Intensiv Ortho-Strips Coarse

DE

Intensiv Ortho-Strips Coarse

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Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C/3
Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C/6
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R/3, rechts
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R/6, rechts
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L/3, links
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L/6, links

GB

Intensiv Ortho-Strips Coarse

Oscillating instruments for a fast and efficient reduction of interproximal enamel in orthodontic treatments.

To be used in combination with the contra-angles:

- Intensiv Swingle, WG-69 LT Ortho, with light
- Intensiv Swingle, WG-69 A Ortho, without light

Alternatively:

- KaVo head 61 LG
- KaVo contra-angle E61C

Product description

- Oscillating diamond-coated and perforated metal strip
- Diamond grit-size: 80 µm
- Available in 3 versions:
 - One-Sided Left (L): diamond coating on the left side of the strip, thickness 0.2±0.05 mm
 - One-Sided Right (R): diamond coating on the right side of the strip, thickness 0.2±0.05 mm
 - Double-Sided: diamond coating on both sides of the strip, thickness 0.3±0.05 mm
- Strip height: 3.7 mm
- Length of the working part: 13 mm
- Sterilizable and reusable

Indication

Unilateral or bilateral interproximal enamel reduction during orthodontic IPR protocol.

Recommendations for use

- Apply Intensiv Ortho-Strips Coarse in interproximal areas according to the IPR enamel reduction protocol for orthodontic treatments.
- A pre-opening of tight contact points using Intensiv Ortho-Strips Opener is suggested, Ref. OS08OP/6, Ref. OS08OP-R/6, right or Ref. OS08OP-L/6, left.
- In case of very tight contact points separate the teeth by placing a wooden wedge cervically. This makes stripping less uncomfortable both for operator and patient.
- Set the contra angle at 20'000 oscillations per minute max.
- Activate water spray during the entire treatment (minimum 50 ml/min).
- Regulate water flow in case of water excess or shortage.
- Insert the strip in a cervical direction while the contra-angle is moving. This prevents the instruments from jamming.
- Contact pressure: bend the strip 1-1.5 mm to achieve the ideal 1-2.5N operating force.
- Do not stop the strip in the interdental space.
- Remove the strip from the interdental space whilst the contra-angle is moving.

Maintenance and Sterilization

- Clean the instruments and remove debris after each use, so as to maintain their abrasive properties.
- Instruments are delivered non sterile. They must be sterilized prior to first use and disinfected, cleaned, and sterilized after each use.
- Use a single-pulsed or fractionated vacuum autoclave and sub-vacuum drying. Chemiclave sterilizers may also be used. Hot air sterilizers are not suitable for diamond instruments.
- Sterilization must be carried out according to validated procedures.

Risk Warnings

- Avoid bending and levering over the limit: use of excessive force (strong levering effect with the contra-angle) could lead to the breakage of the instrument.
- Avoid jamming or levering actions when oscillating, as this increases the risk of instrument breakage.
- Oscillating over 20'000 oscillations per minute could cause breakage of the instrument.
- Avoid temperatures above 180° C which may affect the durability of the instrument.
- Avoid applying forces greater than 3N as this could cause damage to the instrument.
- To ensure traceability of the instruments during their entire application, we recommend keeping the packaging.

Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C/3
Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C/6
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R/3, right
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R/6, right
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L/3, left
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L/6, left

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Instructions for use and safety recommendations

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Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C/3
Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C/6
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R/3, destra
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R/6, destra
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L/3, sinistra
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L/6, sinistra

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Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C/3
Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C/6
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R/3, derecha
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R/6, derecha
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L/3, izquierda
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L/6, izquierda