DE

Intensiv ApproxOpener

Produktbeschreibung

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Intensiv ApproxOpener

Product description

- Manual metal strip
- Versions
 - Serrated
 - Serrated and diamond coated (8µm) on one side
 - Serrated and diamond coated (8µm) on both sides
- Height: 4mm
- Overall length: 20mm
- Thickness of the serrated strip: 0.08mm
- Thickness of the serrated and one-side diamond coated strip: approx. 0.09mm
- Thickness of the serrated and two-side diamond coated strip: approx. 0.10mm
- Total height of the instrument: 18mm
- Total length of the instrument: 27.5mm
- Stainless steel
- Sterilizable and reusable
- Intensiv ApproxOpener is active on one side and passive on the opposite side

Indications

- Opening of tight interproximal areas in orthodontic treatments (one-side and two-side diamond coated versions).
- Removal of black triangles.
- Removal of excess material in the interproximal areas.

Practical Tips

- Intensiv ApproxOpener is positioned incisally/coronally between the teeth in accordance with the interdental space and moved in gingival direction.
- Depending on the initial positioning of Intensiv ApproxOpener in the interdental space, the forward or return movement will be active or passive.
- The removal of excessive restoration material is carried out with the non-diamond coated Intensiv ApproxOpener applied as mentioned above.

Maintenance and Sterilisation

- Intensiv ApproxOpener is provided non-sterile. Before first use it must be sterilized. After the use on the patient the instrument must be disinfected, cleaned and sterilized as well.

Risk Warnings

- It is recommended not to bend the working part or use it as a lever, applying a force superior than 2N, in order to prevent a break of the instrument.
- Please pay attention to the force applied to the instrument when working in the vicinity of soft tissue; apply a force similar to that used with a manual finishing strip to prevent mucosal lesions.
- Intensiv ApproxOpener is held firmly between fingers and secured with dental floss.

Intensiv ApproxOpener

Metal strip serrated
Ref. AO2018/3 Package of 3 Strips

Metal strip serrated and diamond coated on one side
Ref. AO2018OS/3 Package of 3 Strips

Metal strip serrated and diamond coated on both sides
Ref. AO2018DS/3 Package of 3 Strips

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