

# Intensiv Rockle

New

Contra-angle handpiece with more powerful torque for FG instruments

Particularly stable and resistant FG instruments such as the Intensiv ZirconCut are required for the efficient grinding and/or cutting of zirconium oxide ceramic or similarly hard dental materials. Intensiv Rockle was developed to increase the cutting performance of the instruments. The Intensiv ZirconCut abrades with a higher torque than conventional contra-angle handpieces. In addition, the Friction Grip (FG) system has been improved to ensure that the exposed instruments are held securely.

## Product Description

### INTENSIV ROCKLE CONTRA-ANGLE

Intensiv Rockle is a special red ring contra-angle handpiece with standardised ISO coupling, water spray and integrated light. The more powerful internal micromechanics are unique.

- Transmission ratio 1:4, max. 160,000 rpm
- 3 large water spray channels, capacity > 50ml/min
- Longer Friction Grip (FG) system than standard contra-angle handpieces
- Enlarged black push button for easier instrument release and locking



Big black push button



Red ring 1:4



REF WK-900 LT



Film Intensiv Rockle use


## Indications

- Ambitious grinding or cutting of zirconium oxide ceramic or equivalent materials

## Benefits

- 20% stronger torque
- Increased load capacity and better hold of the instruments
- Reduced vibration and noise compared to conventional contra-angle handpieces
- Higher instrument rinsing performance and improved cooling

## Clinical pictures

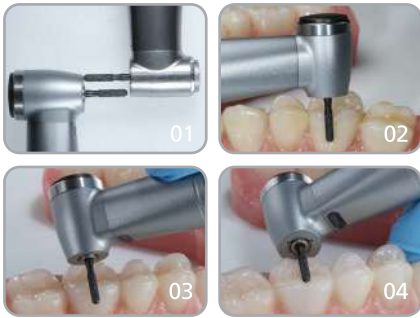

Dr. Alessandro Devigus - Bülach, Switzerland

- 01 Instrument depth compared to conventional contra-angle handpieces
- 02 Buccal crown separation
- 03 Coronal crown separation
- 04 Occlusal crown separation



Intensiv Rockle, REF WK-900 LT

## Intensiv ZirconCut Prosthetics

Intensiv special diamond instruments for more efficient cutting of zirconia ceramic restorations

The Intensiv ZirconCut instruments (see page 30) are particularly suitable for use with Intensiv Rockle. Faster cutting and grinding of zirconium oxide ceramics with the Intensiv ZirconCut / Intensiv Rockle combination. Two products with outstanding performance result in a potentiated performance.

### Benefits

- Significantly faster cutting of zirconia
- Longer and constant higher cutting performance
- Scientific study at the Dental Clinic of the University of Zurich

